GEBRAUCHSINFORMATION INSTRUCTIONS FOR USE

GQ SOLDER STICK

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EN

DESCRIPTION

GQ Solder Stick is a cobalt-based alloy according to ISO 9333:2006 for soldering cobalt-based alloys. The product is intended for dental laboratories for the creation of products that must be rigid and resistant, single crowns, bridges and superstructures on implants.

The alloy GQ Solder Stick was studied to weld cobalt based alloys. It is free of nickel and completely free of toxic elements like beryllium, gallium and indium.

Intended patient population

This alloy is intended to be used for oral rehabilitation in edentulous or partially edentulous patients who suffer from loss of hard dental tissue. There are no limitations related to age, sex, clinical condition or other conditions, other than those possibly determined case by case by the professional user.

Intended users

The product is intended for professionals of the dental sector, in particular: prosthodontic laboratories, dental technicians, dentists. These professional figures shall have a high degree of specialisation and are perfectly aware of the destination of the product and the correct method of use for the realization of a final product and elaborate different methods to reach the best shape of the final prosthesis. Contact and/or manipulation by other people is not envisaged.

INTENDED PURPOSE

Welding of metal prostheses.

MATERIAL CHARACTERISTICS

Color	white
Density	8.2 g/cm ³
Solidus-liquidus temperature	1,071–1,260 °C
Melting point	1,310 °C
CTE value (25–500 °C)	15.5 × 10 ⁻⁶ K ⁻¹
CTE value (25–600 °C)	15.8 × 10 ⁻⁶ K ⁻¹
Mass content of the components (in % by weight)*	Co 62.0; Cr 29.0; Mo 4.0; Si 3.5; C x; Mn x; Fe x
Cytotoxicity test according to to ISO 10993-5	passed

* x = <1 %

PROCESSING INFORMATION

Open flame welding

After having cleaned from the oxid the surfaces to weld, cover them with flux. Heat slowly the parts to be soldered with the flame and lay a homogeneous welding material.

Re-use

The device can be reused until the available material is exhausted.

SAFETY INSTRUCTIONS

- These medical products may only be used and applied in the dental laboratory by qualified personnel!
- In rare cases, the cobalt-chrome alloy can cause skin inflammation when touched by the susceptible persons. A contact test (patch test) is recommended.
- Metal vapours and metal dusts are hazardous to health when inhaled; therefore always use a suction device and/or suitable protective mask!
- The alloy contains Cobalt, element classified as CMR (CAS Number: 7440-48-4).
- Before applying the denture, check whether other metallic dental restorations are present in the oral cavity of the patient. Galvanic effects may occur if different metals are present.
- It is recommended that the patient be made aware of the possibility for dental alloys to affect MRI results.
- Each delivery is identified with a batch number. It is recommended to write it down in patient's file to allow its complete traceability.
- MESA keeps batch traceability for 15 years.
- The storage of the manufactured applications is the responsibility of qualified professionals (dental technicians).
- Users shall report any major accident occurred in relation to the use of this medical device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. A summary of safety and clinical performance (SSCP) is available on Eudamed.
- This product must be stored, transported and used at a temperature between $-30\text{ }^{\circ}\text{C}$ and $+50\text{ }^{\circ}\text{C}$ to ensure the integrity of the packaging and the instructions contained therein..
- No special precautions are required for storage.
- The product is not sterile.

DISPOSAL

The processing scrap must be disposed of as special waste in accordance with the EC directives 2008/98/CEE on waste, and 94/62/CEE on packaging and packaging waste and in compliance with national legislation in force on the subject.

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Gold Quadrat GmbH